


Defibtech DDU-2300 (VIEW) Demonstration Card

Summary

The DDU-2300 Demonstration Card is intended for use by distributors, sales personnel and other users familiar with the DDU-2300 AED to provide an overview of select voice, screen, and text prompts during various rescue scenarios. You can demonstrate pad placement scenarios, the analysis/shock scenario, and CPR coaching.

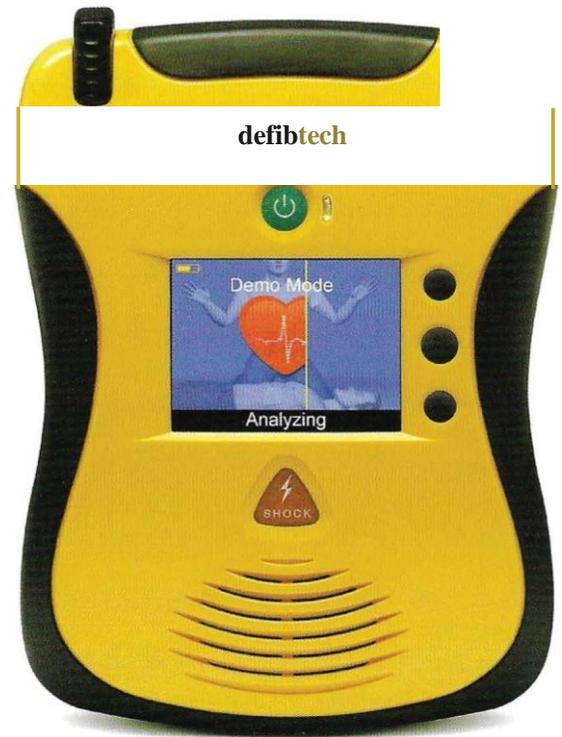

While in Demo Mode, the AED cannot perform a rescue. If at any time during the demonstration there is a need to perform a rescue, press and hold the green on/off button for two seconds to power off the AED. Then press the green on/off button to power on the unit and begin a rescue protocol. The Demonstration Card does not need to be removed in order to perform a rescue.

Demonstration Instructions:

Preparation

1. Make certain that there is a battery in the AED. If a battery is not installed, then insert a battery into the back of the unit. The unit will perform a battery test, which will take a few seconds, and then power off.
2. Make sure the unit is off, then open the data card access door, found on the side of the AED, by slightly pushing and then sliding the door down to release the latch. Insert the Demonstration Card notched end first, label side up, into the thin slot in the side of the AED, centered above the USB port opening, until it clicks in place. Close the data card access door. **NOTE: The Demonstration Card may be left in the AED between demonstrations.**

Demonstration

1. Unplug the pads.
2. Press the center softkey button. The Pads Missing screen will display.
3. Press the top softkey button **to** display the AED status screen.
4. Press the center softkey button along the right side of the display to load the Demonstration Card software. The unit announces "demo mode" and the demonstration will begin. The unit demonstrates how to find the pads and apply them to the patient. If pads remain unplugged after the unit states "apply pads", it will cycle back to "remove clothing from patient's chest" and repeat the preparing the patient for pads application sequence until pads are connected. When the sequence starts to repeat, plug in the pads connector.
5. The unit will demonstrate a simulation of a shockable, patient-connected scenario. The unit will go through the steps of analyzing, charging, and simulated shock delivery. Follow the unit's spoken and displayed instructions until a simulated shock is delivered. (Note: An actual shock will NOT be delivered.)

(continued on next page)

(continued from previous page)

6. The unit will then demonstrate CPR coaching. The following options are available during this phase (Note: These options are **not** available during the final 30 seconds of CPR):

- Press the middle softkey button to receive additional coaching and information. To exit, press the middle softkey button again.
- Press the top softkey button to select whether CPR coaching will include breathing coaching or is "hands-only."

After the CPR phase, the demonstration will return to simulating analyzing the heart rhythm. This time, the demonstration will determine that the patient does not need a shock and the demonstration will continue again with CPR coaching.

7. **If** the pads remain connected, the demonstration will continue to repeat the sequence described in step 6. To repeat the simulation of a shockable scenario, unplug the pads and then reconnect them.

Exiting the Demonstration:

1. Press and hold the green on/off button for two seconds to turn the unit off and end the demonstration.
2. To remove the Demonstration Card, press the card in as far as it will go and then release. The card partially ejects and can then be removed by pulling the card the rest of the way out. (Note: You may leave the Demonstration Card in the unit for your convenience in performing future demonstrations.)



The OOU-2300 AED should always be ready for clinical use after a demonstration. Make sure the battery pack is sufficiently charged and rescue defibrillation pads are properly installed so the AED can continue to perform its automatic self tests to help ensure that it is ready for emergency use.

To Order Call:

AEDSuperstore®

World's Largest Automated External Defibrillator Source

800.544.0048
help@aed.com

www.AEDSuperstore.com

1800 US Hwy 51 N | Woodruff, WI 54568