



cprcube will help save your family's lives

To Order Call:

AEDSuperstore[®]

World's Largest Automated External Defibrillator Source


800.544.0048
help@aed.com

www.AEDSuperstore.com

1800 US Hwy 51 N | Woodruff, WI 54568

cprcube[™]

My Family's Life Saver

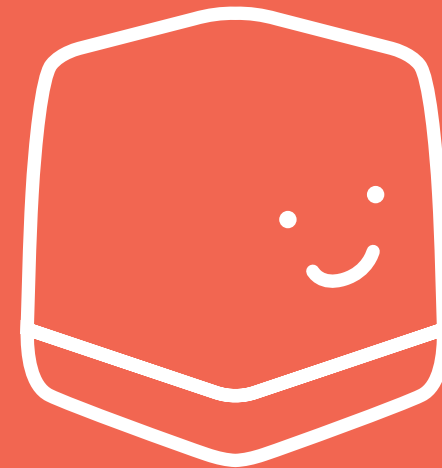


www.cprcube.com



Essential.

cpr**cube** is a Hands-only CPR training aid that assists anyone to master chest compression without using expensive CPR manikins.

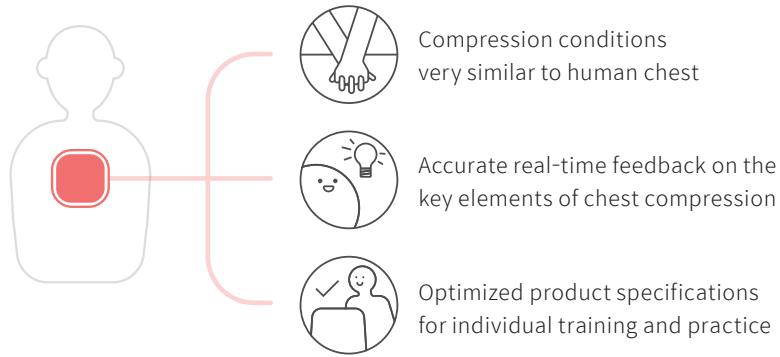

Everywhere.

Any place can be a first aid training site with the cpr**cube**. Just open the box and expand it. You can learn CPR from WHEREVER and WHENEVER you want!



Exciting.

cpr**cube** enables users to have fun, immersive education through visual, auditory, and tactile feedback.

Real-time feedbacks

1. SOUND

cpr**cube** makes sound when the appropriate compression depth (5-6 cm) and recoil (less than 1cm) are applied. If it is difficult to apply the right speed, please press the cube for 3 seconds. Then the 100cpm sound guide will be given.

2. LED LIGHT

Successful compression (depth, recoil, rate) makes LED lights flow. LED lights up one by one for each successful compression. If you succeed 15 times in a row, LED revolves around the cube to express the blood flow.

3. VIBRATION

Feel a heartbeat by repeating proper chest compressions.

cpr**cube** gives vibration when 15 successful compressions are applied in a row.

Analysis of user's compressions via sensors



Specifications	
Dimension	100×95×95 (mm)
Weight	120g (without battery)
Power	4 AA batteries, 3V

