

this document brought to you by
AED Superstore

ZOLL Medical Mobilize Rescue Systems

ZOLL Medical's Mobilize Rescue Systems have been designed to give the layperson the ability to respond to nearly any emergency situation which may arise in today's society.

800.544.0048
AEDSuperstore.com



IMPORTANT: Please follow these instructions carefully when setting up your Mobilize Trauma Kit.



Mobilize™ Trauma Kit

Quick Start Guide

Comprehensive • Mobile page 2
Public Access • Compact page 6

Equip. Empower. **Respond.**

ZOLL®

DISCLAIMERS. ALTHOUGH THE PRODUCTS MAY BE USED BY THE INDIVIDUAL TO PROVIDE MEDICAL ATTENTION, (A) MOBILIZE IS NOT A HEALTH CARE PROVIDER AND IS NOT RENDERING MEDICAL ADVICE OR DIAGNOSIS TO ANYONE; AND (B) THE PRODUCTS ARE NOT INTENDED TO REPLACE THE NEED FOR SEEKING MEDICAL ATTENTION FROM A MEDICAL PROFESSIONAL.

800.544.0048 www.AEDSuperstore.com

Comprehensive or Mobile kits



Box contents:


- Mobilize Trauma Kit
- Quick Start Guide (this document)
- Charging equipment (USB cable and wall adapter)
- Security seals and inspection card

Requirements:

- Access to WiFi

Kit contents

A1	Tourniquet	D1	Face Shield
B1	QuikClot®	E1	Allergy Medicine
B2	Pressure Dressing	E2	Aspirin
C1	Chest Seal	E3	Glucose

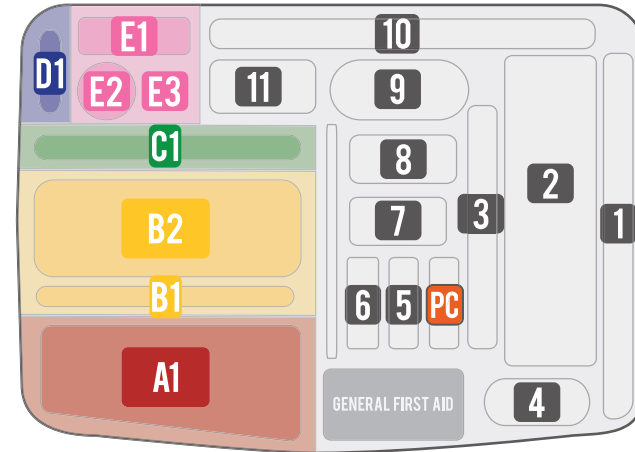

1	Splint
2	Trauma Gauze
3	Gauze
4	Rolled Gauze
5	Biohazard Bag
6	Hand Wipes
7	Emergency Blanket
8	Ice Pack
9	Elastic Wrap
10	Burn Dressing
11	Triangular Bandage
PC	Portable Charger*

***COMPREHENSIVE KIT ONLY**

ON LID	
Gloves	Trauma Shears
Tape	Incident Pad & Pen




www.zoll.com

Equip. Empower. [Respond.](#) 3

800.544.0048

www.AEDSuperstore.com

Setting up the included tablet

- 1) Turn the tablet on by pressing and holding the side button on the right until the tablet turns on (approximately 5 seconds).
- 2) After the Mobilize Rescue application launches, press “ACTIVATE APP.”
- 3) Press the “SETTINGS”  button in the upper right-hand corner.
- 4) Press “Change WiFi” and toggle from Off to On.
- 5) Connect to your local WiFi network by selecting the network name and entering the password.
- 6) Press the back button  twice, then press the “ACTIVATE APP”  button to reach the Login/Register Page.
- 7) Press Register to create an account; or enter your existing account information, press “LOGIN” and skip to step 9.
- 8) Enter your new account information: First name, Last name, email, and password. The password must be at least 8 characters. Check the box “I have read and agreed with the Terms and conditions.” Press “SUBMIT.”
- 9) On the next page under “Activate app with,” choose either:
 - a. License Code: provided by your local authorized ZOLL representative
 - b. Serial Number: Located near the hinge on the Comprehensive carrying case, and on the back of the tablet for Mobile units.
- 10) Press Submit and you will see “Take a Deep Breath, Mobilize Will Help You.”
You are now ready to use Mobilize.

- 11) Use the charging equipment to charge the tablet’s battery to 100%. To ensure that the kit is ready if needed at a moment’s notice, charge the battery once per month.
 - The Comprehensive kit can be charged with the case closed. The tablet goes to sleep when lid is shut and wakes up when it is opened.
 - To put the Mobile kit to sleep, press the side button on the tablet. Double tap the screen to wake it up.

Please contact your Authorized ZOLL Distributor to purchase replenishments for utilized or expired supplies.

Public Access or Compact kits



Box contents:

- Mobilize Trauma Kit
- Quick Start Guide (this document)

Requirements:


- Mobile phone or tablet for app installation
- Access to WiFi or cellular connection

Kit contents

A1	Tourniquet	B2	Pressure Dressing
B1	QuikClot®	D1	Face Shield
C1	Chest Seal	7	Emergency Blanket



Setting up the mobile application

- 1) Go to the Apple App Store or Google Play Store and search for the Mobilize Rescue application.
- 2) Download the Mobilize Rescue application to your device.
- 3) When the download is complete, open the application.
(Apple users: if prompted to allow notifications, press “Allow.”)
- 4) After Mobilize launches, press the “ACTIVATE APP”  button.
- 5) Press “REGISTER” to create an account; or enter your existing account information, press “LOGIN” and skip to step 7.
- 6) Enter your new account information: First name, Last name, email, and password. The password must be at least 8 characters. Check the box “I have read and agreed with the Terms and conditions.” Press “SUBMIT.”
- 7) On the next page under “Activate app with,” choose either:
 - a. License Code: provided by your local authorized ZOLL representative
 - b. Serial Number: For the Public Access kit, the serial number is located on the front of the carrying case. For the Compact kit, use the number in the box here:

- 8) Press Submit and you will see “Take a Deep Breath, Mobilize Will Help You.” You are now ready to use Mobilize.

If you do not have a License Code, or wish to get access to the application for additional users, please contact your Authorized ZOLL Distributor.

Thank you for taking safety to heart and purchasing your Mobilize Trauma Kit (Mobile, Comprehensive, Public Access, or Compact). The Mobilize system is designed to provide lay rescuers with equipment and guidance to help save lives. Each kit includes medical and trauma supplies, and is paired with an interactive application that provides real-time instructions on how to use the supplies to provide care for injured or sick victims. The step-by-step directions provided through the Mobilize Rescue application enables bystanders to assess, monitor and manage medical emergencies in the critical minutes before EMS arrives.

If you have any questions or concerns, please contact ZOLL Technical Support at 1-800-348-9011, or email us at techsupport@zoll.com; or contact your authorized ZOLL distributor.

For assistance, contact ZOLL Technical Support at
800 348-9011
8:30am - 6pm Eastern Time
or techsupport@zoll.com



[@ZOLLPublicSfty](https://twitter.com/ZOLLPublicSfty)



[ZOLL Public Safety](https://www.facebook.com/ZOLLPublicSafety)



[ZOLL Medical](https://www.linkedin.com/company/zoll-medical)



978-421-9655



info@zoll.com

Copyright © 2021 ZOLL Medical Corporation. All rights reserved.

www.zoll.com

9650-001835-01 Rev. A

800.544.0048 www.AEDSuperstore.com